



# AURA™

The first automated, point-of-care disinfection device utilizing powerful disinfectants in a patented new process. Broad material compatibility with broad-spectrum efficacy. Zero occupational exposure to chemical disinfectants. Improved sustainability.

AURA-D Cartridge (EPA Reg. No. 93728-2) when used with the AURA System is a hospital disinfectant for hard non-porous surfaces. The AURA-D Cartridge when used with the AURA System is effective against SARS-CoV-2 (Coronavirus), the virus that causes COVID-19 and can be found on EPA List N, Disinfectants for Coronavirus (COVID-19).

# Advancing Patient Safety

Then



Hospital linens were frequently found to be a source of contamination causing outbreaks of yellow fever, typhoid and TB<sup>1</sup>



Now



Modern hospital laundries use scientific advancements in water treatment, chemistry and process control to achieve up to a 6-log reduction



Until the 1940s the effects of scattered radiation were poorly understood, leading to injury and fatalities for patients and healthcare workers<sup>2</sup>



The application of improved technology has shielded both patients and clinicians from radiation hazards



Studies show disinfectants pose health & environmental risks. Current practices are costly, time consuming & can be ineffective when directions for use are not followed.



**AURA™** revolutionizes the way non-critical medical devices and point of care items are disinfected

Our cost effective monthly subscription includes **AURA**, **AURA-D** cartridges, service and maintenance

# The AURA™ Process

## Convenient Plug-and-Play Set-Up

Plug into any standard outlet. Insert the **AURA-D** cartridge and you're ready. No venting required; only clean air remains.



## Mix & Match

Place combinations of non-critical devices, equipment, and/or personal items inside.



## Just Push One Button-It's Automated

Simply close the door and push start. **AURA** does the rest using our patented application of activated oxygen and hydrogen peroxide.



## Walk Away

No need to monitor contact time. **AURA** automatically achieves total disinfection and visually signals when the cycle is complete.



## Exposure Free

Keep yourself and your patients safe from chemical exposure. **AURA** automatically neutralizes the disinfectant.



## Multiple Organisms

No more guessing about which wipe to use. **AURA** controls listed microorganisms with broad material compatibility.




## Automated

**AURA's** process is fully automated with visual indicators and no audible alarms.



## No Residue

**AURA** won't damage your equipment or cloud device screens.

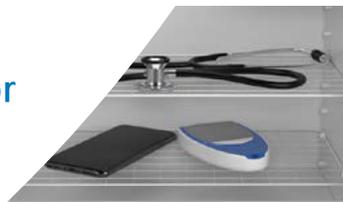

**Automated Quality Controlled Disinfection using processes similar to SPD**

# Introducing . . . **AURA™**

**AURA** disinfects using process controls and efficacy validation similar to those found in SPD.

## Disinfection . . . done right



4020 Lake Washington Blvd. N.E.  
Suite 300, Kirkland, WA 98033  
[www.sterifre.com](http://www.sterifre.com)

**AURA-D** Cartridge is only for use in conjunction with the **AURA** Automated Disinfection System.  
**AURA** Automated Disinfection System is intended to disinfect hard non-porous surfaces of reusable non-critical medical devices and general-use items used in healthcare facilities.

<sup>1</sup>Barnes, M. E., "A Public Health Role for the Laundry," American Journal of Public Health, 35, 1945, pp. 1277-1281, Available online at: <https://ajph.aphapublications.org/doi/pdf/10.2105/AJPH.35.12.1277>

<sup>2</sup>Brodsky, A., Kathren R., "Historical Development of Radiation Safety Practices in Radiology," RadioGraphics, 9(6), 1989, pp. 1267-1275, Available online at: <https://pubs.rsna.org/doi/pdf/10.1148/radiographics.9.6.2685944>